


# QUICK REFERENCE GUIDE FOR THE SELF-TEST

Product: NASOCHECKcomfort SARS-CoV-2 Rapid Antigen Test  
 Manufacturer: Beijing LEPU Medical Technology Co., Ltd.

## First steps (All components are at room temperature)

**1**

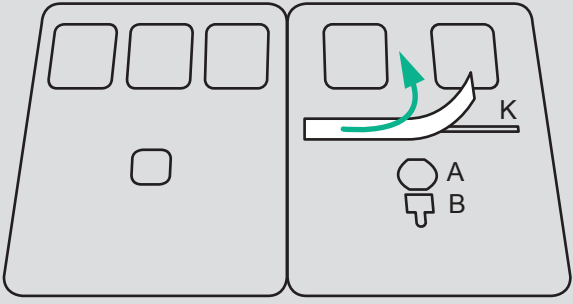


- Remove the foil on the adhesive strip "K"
- The test card must be used within one hour after opening the sealed pouch

## Sample collection

**2**


Into each nostril:  
 Insert tip of the swab approx. 2 - 3 cm and rotate it five times while maintaining contact with the nasal mucous membrane

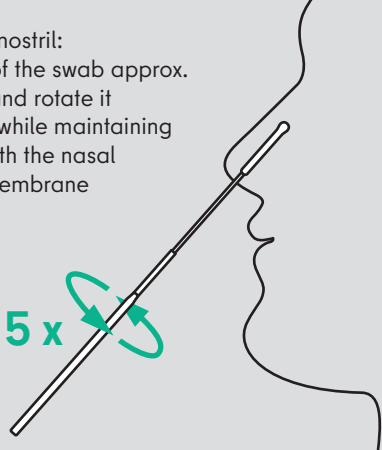

5 x


## Treatment of the sample

**3**

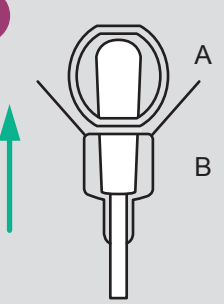

Push swab through opening "B" into well "A"

**4**

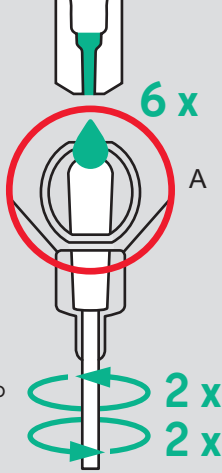

Add six drops into sample well "A"


Then rotate swab twice in each direction

6 x

2 x

2 x


**5**

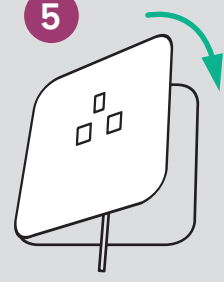

Close card, stick it together with adhesive strip "K" and carefully press together

**6**


Read result **after 15 min.** on the front of the card



Don't analyse any result after more than 20 minutes!

## Analysis of the results

Every colour shade at the test line (T) is considered positive

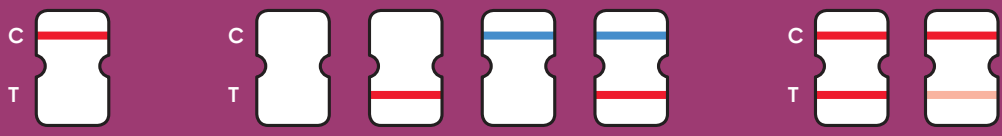


**Negative**  
 Coloured control line (C)

**Invalid/unclear**  
 No control line (C)

**Positive**  
 Coloured control (C) and test line (T)

## Disposal



Carefully pack used test kits and dispose in domestic waste

Pupils and school and daycare staff can go to school/daycare centre

Repeat test with new material

- Parents report child as sick to the school, or school or daycare staff call in sick
- Pupils and school and daycare staff have PCR follow-up testing performed at one of the central test centres



Test centres for PCR follow-up testing