

QUICK REFERENCE GUIDE FOR THE SELF-TEST

Product: Medicovid-AG® SARS CoV-2 Antigen Nasal Self-test
 Manufacturer: Medice Arzneimittel Pütter GmbH & Co. KG


First steps (All components are at room temperature)

1 Blow nose several times and then wash hands

Place test tube in a clamp (e.g., clothes peg)

2 Fill with 8 drops (0.25 ml) of buffer solution


8x

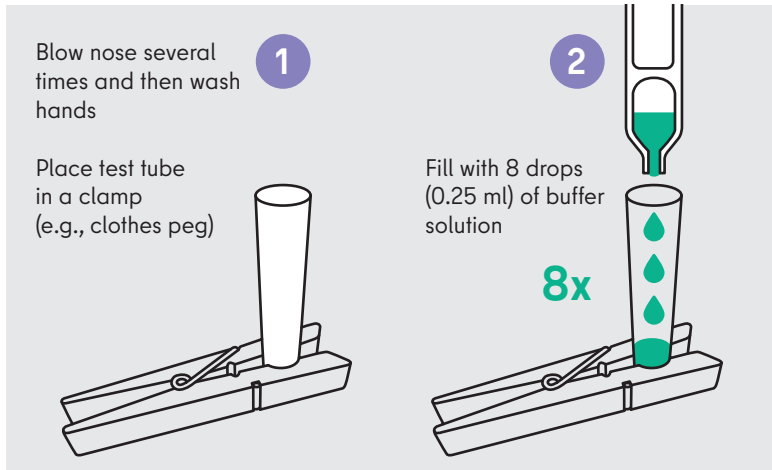

Sample collection

3 Insert tip of swab 2.5 cm into each nostril, rotate it three to four times while at the same time moving it over the nasal mucous membrane, then leave swab in the nostril for several seconds

3-4x

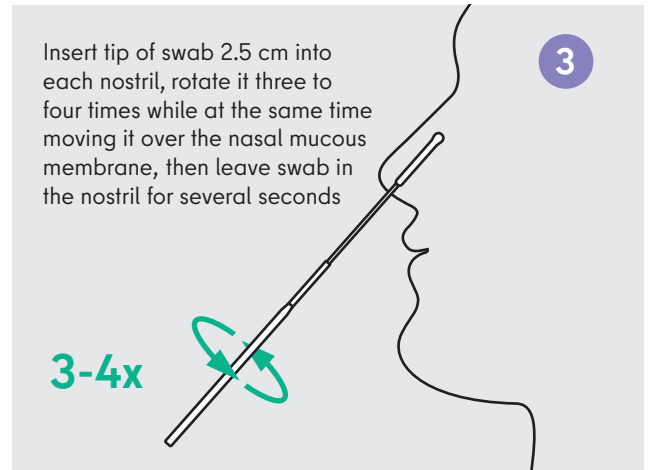

Treatment of the sample

4 3-5x Insert swab, rotate it three to five times and press it against the sides of the test tube

5 Leave the swab for one minute in the test tube

6 Press test tube against the swab to dislodge antigens

Then remove swab and place dropper cap on tightly


7

8 Add two to three drops to the sample well "S" on the test cassette

2-3x


9 Read after 15 min result

Don't analyse any result after more than 20 minutes!

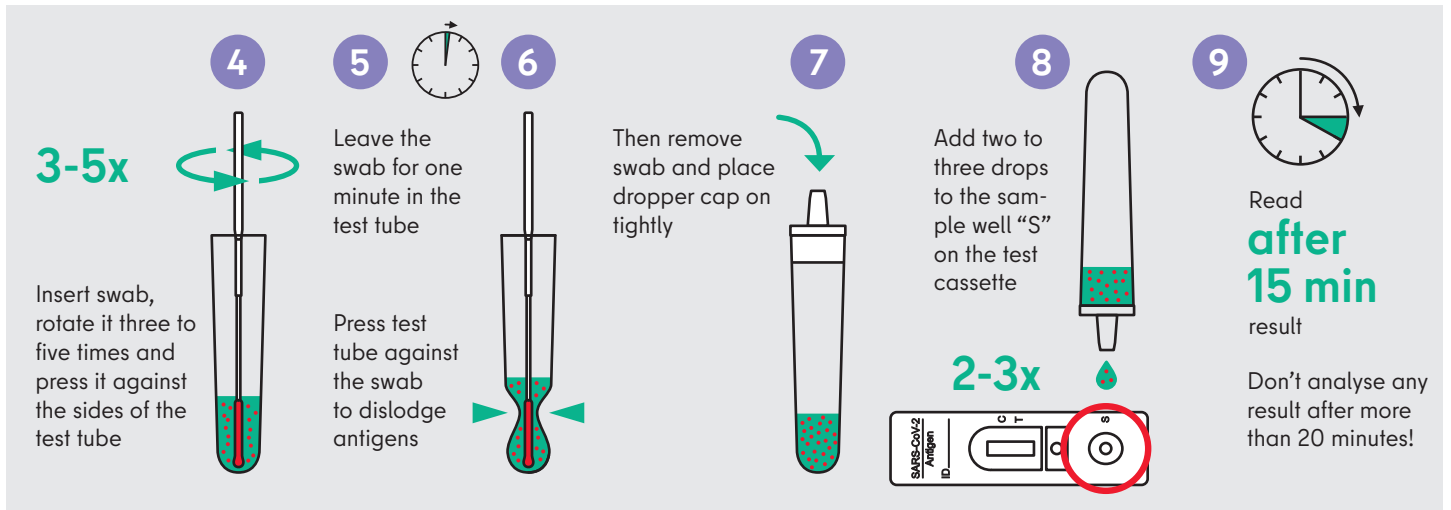

Analysis of the results

Every colour shade at the test line (T) is considered positive

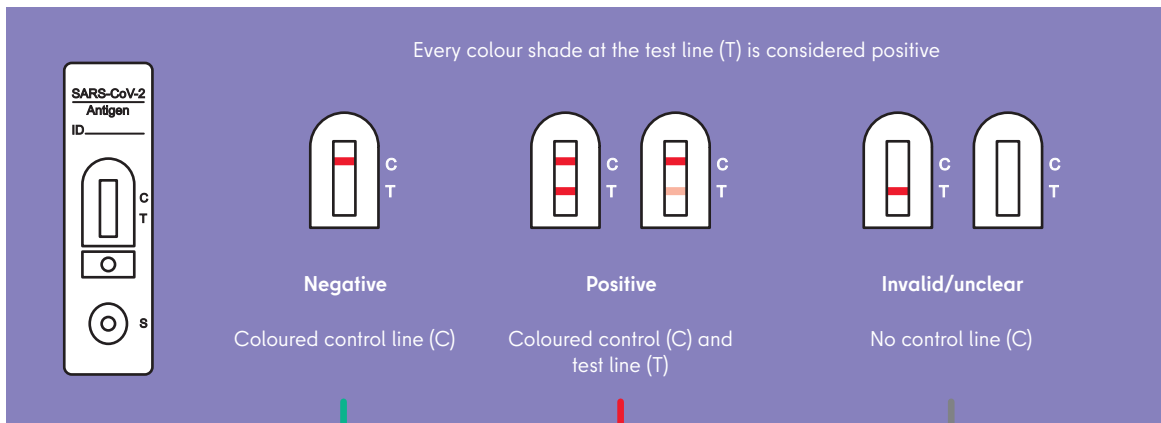

Negative
Coloured control line (C)

Positive
Coloured control (C) and test line (T)

Invalid/unclear
No control line (C)

Pupils and school and daycare staff can go to school/daycare centre

- Parents report child as sick to the school, or school or daycare staff call in sick
- Pupils and school and daycare staff have PCR follow-up testing performed at one of the central test centres

Repeat test with new material

Disposal



Carefully pack used test kits and dispose in domestic waste